



Interdental-Aid



Stijn Teysen, DVM
Beekstraat 14
1730 Asse
Belgium
www.stijnteysen.be
info@stijnteysen.be
+32 (0) 495 93 28 04



The Interdental-Aid device has been tested repeatedly in practice and the producers are convinced that it will increase your working comfort as well as the quality of your work. The device was developed for the treatment of diastemata and periodontitis in horses in general and the filling of diastemata with vinyl polysiloxane in particular.

The Interdental-Aid was designed and produced with 3D technology and laser sintering. The production procedure and material used for this product is chosen for its strength, durability and easy maintenance.

No special maintenance is needed. Dirt can be removed with a damp cloth. The device is splash-proof, but should not be submerged in water.

The device has to be used judiciously. The use of unnecessary force or dropping the device must be avoided. The Interdental-Aid is developed for use on a sedated horse with presence of a properly inserted mouth speculum. The risk of sudden movements or use of excessive force will therefore be limited.

The producer declines any responsibility for possible accidents deriving from the (erroneous) use of the device.

The producer also declines any responsibility in possible damage to the device, the patient or other parties as a consequence of wrong use, wrong assembling or use for purposes the device was neither destined nor tested for.

Please note that the design is patented and that copying is forbidden by law.

Manual of use

Before using the Interdental-Aid, the device must be properly assembled.



The pestles are of different thicknesses and must be inserted correctly (see image).

If inserted correctly, both pestles can be moved easily. They can only be removed through the pistol side. The pestles can therefore not fall out during the change of cartridges while treating a patient.



The extension piece must be placed into the pistol correctly.



On top of the Interdental-Aid, you will see the mention 'NO V-groove'. Hence the V-groove on the cartridge must be placed on the other side.



Careful: if you insert the cartridges incorrectly, the pestles will block and can prevent you from removing the cartridge.



Once the Interdental-Aid is assembled and the pistol placed, you will be able to insert a cartridge on the side. Move the lid up to lock the cartridge, the device will be ready for use.



The cartridge and/or Interdental-Aid can only be removed from the pistol once the pestles of the pistol are completely withdrawn.

No force is required for the replacement of the cartridges or the placing/removal of Interdental-Aid. Please check if you are assembling the device correctly if you have trouble placing or replacing cartridge or pistol.